

OWNER'S MANUAL

Read all instructions and warnings before using the StrideLight™ Ultrabright Walking Cane. If you are unable to understand these instructions, contact a healthcare professional before using this product. Otherwise injury or damage may occur.

To adjust the height of the StrideLight™ Cane

1. Loosen the locking collar by turning counterclockwise.
2. Press the gold snap button inwards.
3. Slide the lower segment up or down until proper height is reached.
4. Adjust the lower segment so that the gold button makes an audible CLICK when it is fully extended through the adjustment hole and is securely in place.
5. Tighten the locking collar by turning clockwise.

To change the batteries located in the handle portion of the StrideLight™ Cane

Remove the end-cap by pushing in and turning the cap counterclockwise one-third turn. Remove and replace used batteries. Replace the end cap by seating between the threads, push in and turn clockwise until the cap locks in place. The light intensity of the StrideLight™ light will decrease when there are approximately 15 minutes of remaining battery life, indicating that it is time to replace the existing batteries with 2 fresh AA batteries. Alkaline batteries may be used, however longest battery life will be achieved with lithium batteries.

Caring for the StrideLight™ cane

Clean with a damp cloth and dry thoroughly before use, including the cane tip. Replace cane tip or foam grip if damaged or worn.

Warnings

- Always consult a physician or therapist before using this device so that it is properly adjusted and you understand its proper use. This device is intended to provide additional lighting in the immediate walking environment under conditions of low lighting.
- Do not touch or scratch the LED as this may cause irreparable damage not covered by the warranty.
- Be sure that the gold button is securely in place and that the cane is adjusted to the proper height before using.
- Do not mix alkaline, lithium or rechargeable batteries.
- Frequently check the rubber tip for cracks, wear or tears. Replace immediately if tip is worn or damaged.

Do not look directly at the light or point the light directly at others.

Turn light off when not in use.

Do not immerse the StrideLight™ Cane in water.

The StrideLight™ Cane is designed for use by individuals of up to 250 lb. to provide enhanced environmental lighting and walking support. **It is not intended to support an individual's total weight.**